# FEATHER MICROTOME HOLDER

## No.130C-A

## No.130C-S

### Instructions for use


www.feather.co.jp/en/m\_products/  
www.pfmmedical.com/downloads  
eIFU is available in: English, Deutsch, Français,  
Español, Italiano, Português, Nederlands


#### The device's intended purpose

- This device is intended to be attached to the dedicated microtome blade and used for thin-sectioning of tissue that has been processed for pathological examination.
- Dedicated blade: FEATHER MICROTOME BLADE, Low Profile (with the exception of S35L and S35LL)
- This device is intended for laboratory professional use.

#### Contents

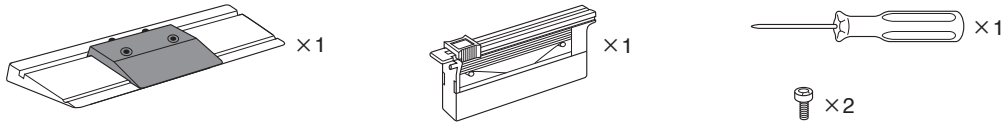
#### How to clamp the disposable microtome blade
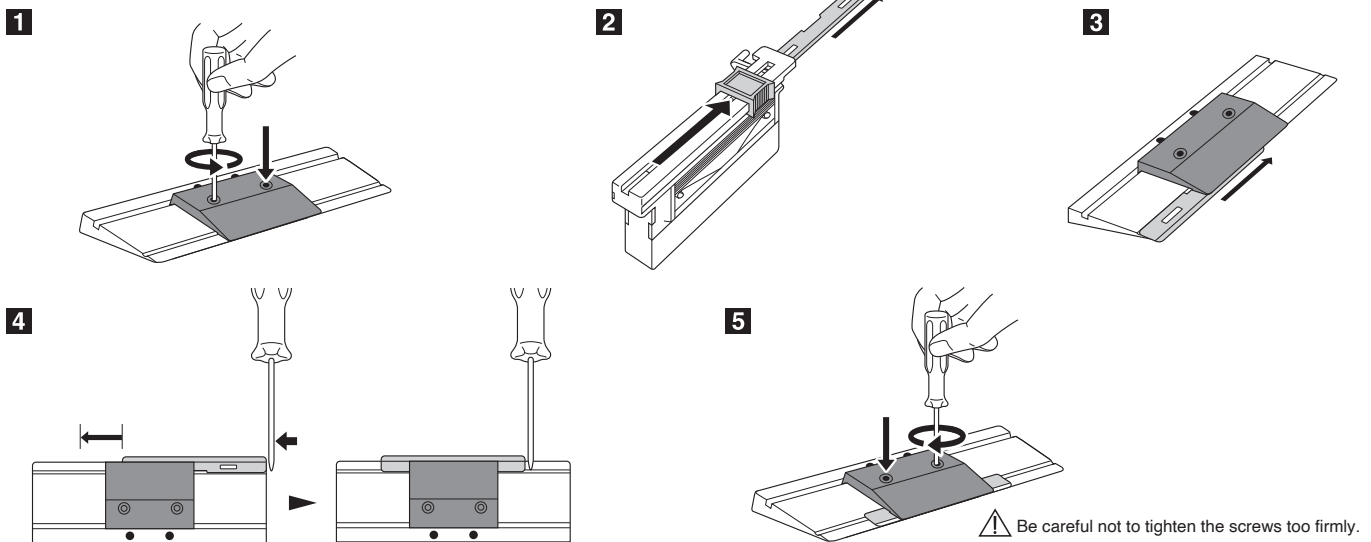
#### Caution


- Precautions for use**
1. The products are only for medical use. Disposable microtome blades are extremely sharp, so please use them with care. Used microtome blades can be safely kept in the part of the dispenser marked accordingly.
  2. To insert a new disposable microtome blade, please use the insertion device on the microtome blade holder or dispenser.
  3. Make sure to fix the microtome blade holder at a clearance angle of at least 5°.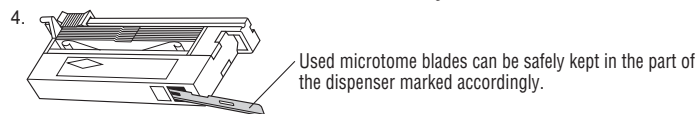

5. The disposable microtome blades have been treated with anti-corrosion oil. This will not cause any problems for the sectioning process, so removal of the oil is not necessary.
6. Disposable microtome blades will quickly become dull if used for paraffin blocks which have not been well infiltrated with paraffin, for sectioning non-decalcified tissues or tissues having minute structures.

#### Safety and care instructions

1. Wipe the surface of the microtome blade holder daily using machine oil. Although the holder is made of alloyed tool steel and is rust-resistant, moisture and fingerprints could cause rust.
2. Handle with care in order not to damage the precision ground edge of the microtome blade holder.
3. Prior to storing the microtome blade holder for a longer period of time, fully disassemble all the parts and apply some oil to all surfaces, including the grooves, fixing plates and screw holes.
4. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

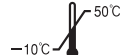pfm medical gmbh  
Wankelstr. 60, 50996 Köln, Germany  
T +49(0)2236 9641-0, F -20  
www.pfmmedical.com

FEATHER SAFETY RAZOR CO., LTD.

3-70, OHYODO-MINAMI 3-CHOME, KITA-KU, OSAKA 531-0075, JAPAN  
www.feather.co.jp/en

MADE IN JAPAN

#### Symbols glossary

	Manufacturer		Catalog number		Keep away from sunlight		Consult instructions for use		Batch code		Do not use if package is damaged and consult instructions for use
	Authorized representative in the European Community/European Union		Importer		Keep dry		Caution		Temperature limit		In Vitro Diagnostic Medical Device


# FEATHER MICROTOME HOLDER ANTI-ROLL DEVICE SET


www.feather.co.jp/en/m\_products/  
www.pfmmedical.com/downloads  
eIFU is available in; English, Deutsch, Français,  
Español, Italiano, Português, Nederlands

## Instructions for use


### Contents

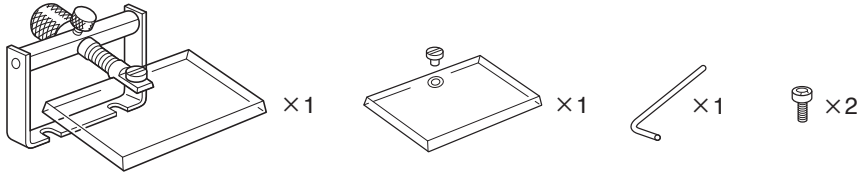
### Setting up the anti-roll device
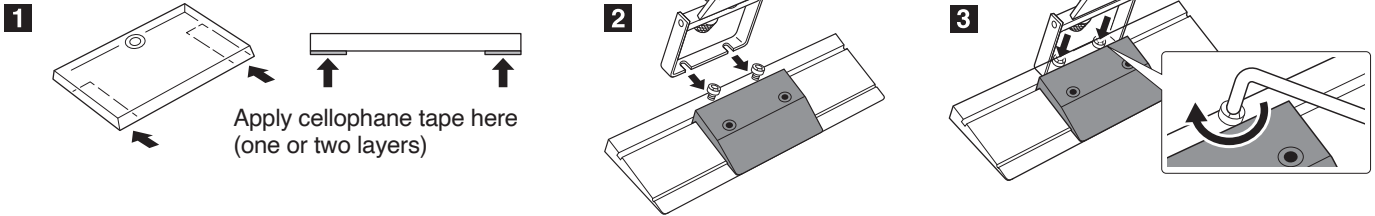
### Adjusting method

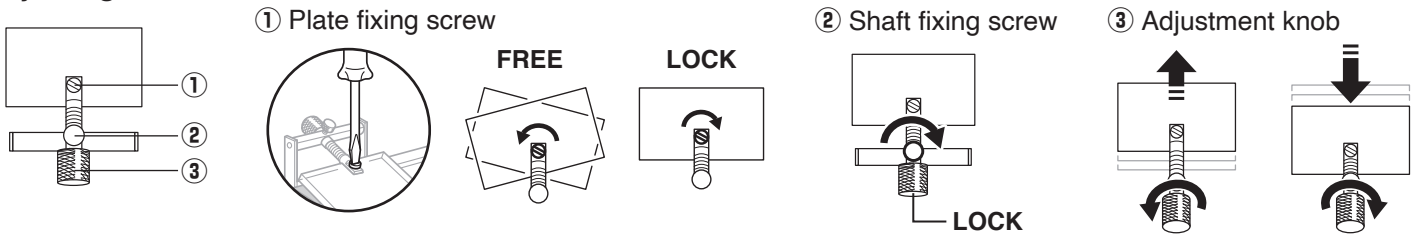
### Before adjustment

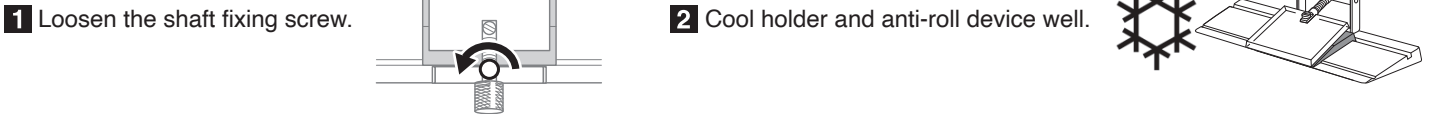
### Adjustment


1 Adjust the anti-roll plate parallel to the blade.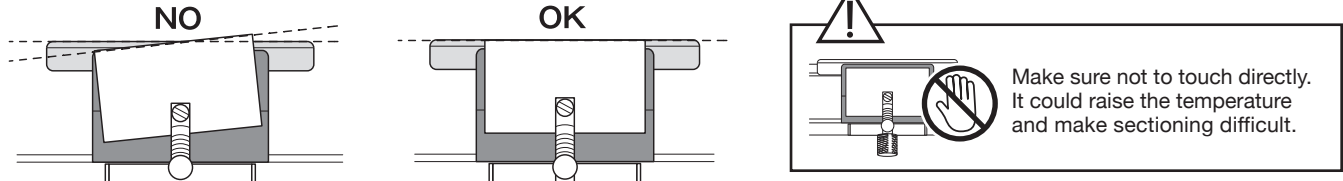


2 Set the anti-roll plate from position 1 to 2.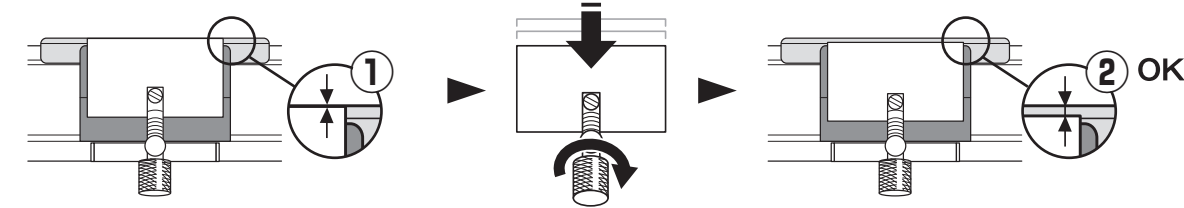

3 Gradually move the anti-roll plate forward to position 2.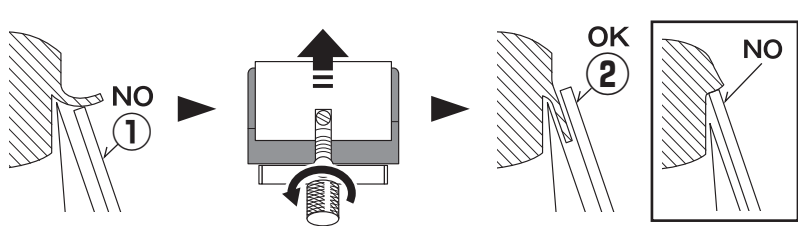

4 When the proper position of the anti-roll plate is found, tighten the shaft fixing screw.

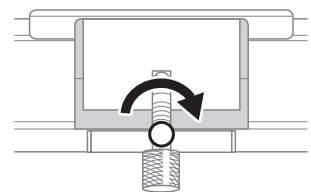pfm medical gmbh  
Wankelstr. 60, 50996 Köln, Germany  
T+49(0)2236 9641-0, F-20  
www.pfmmedical.com

**FEATHER SAFETY RAZOR CO., LTD.**  
3-70, OHYODO-MINAMI 3-CHOME, KITA-KU, OSAKA 531-0075, JAPAN  
www.feather.co.jp/en **MADE IN JAPAN**

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