# FEATHER® FEATHER SAFETY TRIMMING KNIFE HANDLE F-1326WP

## Instructions for use

### The device's intended purpose

- This device is intended to be attached to the dedicated FEATHER SAFETY TRIMMING KNIFE BLADE and used for dissection of tissues.
- Dedicated blade: FEATHER SAFETY TRIMMING KNIFE BLADE
- This device is intended for laboratory professional use.

### Device description

The FEATHER SAFETY TRIMMING KNIFE HANDLE is a reusable handle made from stainless steel and wood-plastic composite and used with FEATHER SAFETY TRIMMING KNIFE BLADE which are specially designed for this product.


### Precautions for use

- Always check the lock screw is connecting to the hole of the replacement blade before use.
- Make sure the blade is fixed on the handle before use.
- Use only as a trimming instrument.
- Do not use when there is something wrong with the product.
- For prevention of contamination, dispose properly after use.
- The color of the handle may fade due to cleaning, disinfection, or sterilization, but it does not affect the functionality of the product.


### How to set the blade

Do in reverse order when detaching the blade.

⚠ Use care when handling blades.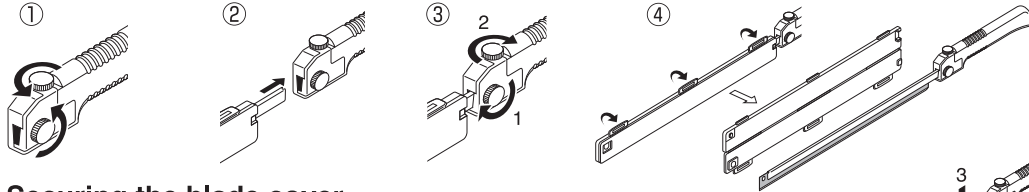

### Securing the blade cover

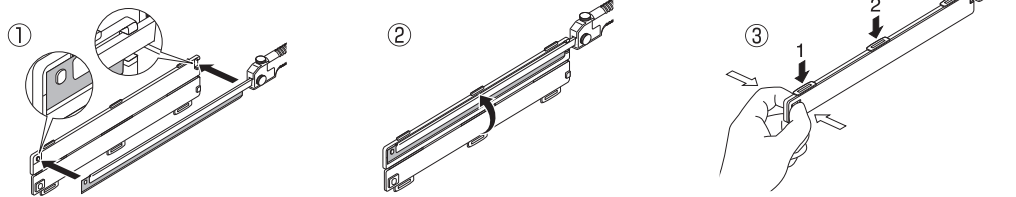
### Opening and cleaning the safety trimming handle

To assemble, carry out these steps in reverse order.

⚠ Before taking the handle apart, carefully remove the blade.

⚠ Take care not to lose any of the screws.

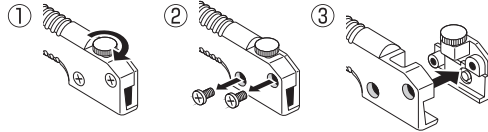### Resterilization instruction

#### Warnings

- Make sure to check if the device is in proper condition for reuse after reprocessing.
- Do not clean the handle while the blade is attached.

#### Limitations on reprocessing

- In order to maintain the usefulness of the device, clean and sterilize the device immediately after use following the reprocessing instructions, and store properly until next use.
- The product life depends on the status and frequency in which the product is being used.
- The device must be handled only by trained and experienced personnel.

#### Point of use

Wash off debris on the device with purified water (<40°C/104°F) immediately after use.

www.feather.co.jp/en/m\_products/  
www.pfmmedical.com/downloads  
eIFU is available in; English, Deutsch, Français,  
Español, Italiano, Português, Nederlands

### Transportation

Make sure to prevent the product from damaging anything, or being damaged by any other materials during transportation.

### Preparation for cleaning

Remove the blade from the handle before cleaning.

### Cleaning

Make sure to prevent the product from damaging anything, or being damaged by any other materials.

Usable detergents: Neutral detergent, Neutral enzyme detergent, Alkaline detergent (pH>10), Enzyme alkaline detergent (pH>10)

Preparation of detergent: Follow the instructions of the detergents, dilute to specified concentration.

Temperature: 40°C-50°C/104°F-122°F

#### <Immersion cleaning>

1. Immerse the product in detergent for 15 to 20 minutes. If the device is tainted badly, increase the immersion time.
2. Rinse the device with purified water to remove detergent thoroughly.

#### <Immersion cleaning with the ultrasonic bath>

1. Put detergent into the ultrasonic bath.
2. Immerse in the ultrasonic bath and process in 28-40kHz for 3 minutes.
3. Rinse the device with purified water to remove detergent thoroughly.

#### <Washer-Disinfector>

1. Put the device on an instrument tray.
2. Put instrument tray into the washer-disinfector and start the cycle.

### Drying

Dry the product thoroughly under clean conditions.

### Packaging

Pack the product into a sterilization container or appropriate sterilization bag.

### Sterilization

Sterilize the device with autoclave.

Recommended parameters for sterilization: 134-135°C/273-275°F, 3-5 minutes

<In case of cleaning for Creutzfeldt-Jacob/BSE contaminated instruments.>

Recommended parameter for deactivation of prion: Autoclave with; \*1 134-135°C/273-275°F, 8-10 minutes \*2 134-135°C/273-275°F, 18 minutes

\*1 Sterilization parameter in case of using washer-disinfector with alkaline detergent(90-93°C).

\*2 Sterilization parameter recommended by The Robert Koch Institute when an alkaline cleaning solution should not be used.

### Storage

Store the device appropriately until next use following the rules each facility sets for the method and conditions of storage after sterilization.

### Additional remarks

The instructions provided above have been validated by the medical device manufacturer as being CAPABLE of preparing a medical device for re-use. It remains the responsibility of the processor to ensure that the processing as actually performed using equipment, materials and personnel in the processing facility achieve the desired result. This requires validation and routine monitoring of the process. Likewise, any deviation by the processor from the instructions provided should be properly evaluated for effectiveness and potential adverse consequences. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

**FEATHER SAFETY RAZOR CO., LTD.**  
3-70, OHYODO-MINAMI 3-CHOME, KITA-KU, OSAKA 531-0075, JAPAN  
www.feather.co.jp/en **MADE IN JAPAN**

### Symbols glossary

	Manufacturer		Catalog Number		Keep away from sunlight		Consult instructions for use
	Authorized representative in the European Community/European Union		Importer		Keep dry		Caution
	Batch code		Do not use if package is damaged and consult instructions for use		Temperature limit		In Vitro Diagnostic Medical Device

# FEATHER SAFETY TRIMMING KNIFE HANDLE TRIMMING-GRIFF F-1326WP

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